EDENTA AG, Hauptstrasse 7, CH-9434 AU/SG – **Switzerland**
Tel.: +41 71 747 25 25 Fax: +41 71 747 25 50 e-mail: info@edenta.ch

EDENTA Etabl., Industriestrasse 13, LI-9486 Schaanwald – **Liechtenstein**
Tel.: +423 375 20 50 Fax: +423 375 20 55 e-mail: info@edenta.com

DE

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GB

Instructions for endodontic instruments:

When using hand type instruments please put on the safety chain for root canal instruments or work with rubber dam. When using a hand type endo instrument in an endo contra-angle (Giromatik contra-angle), please follow the manufacturer's instructions.

Peeso, No. 183.15... and 183.19... :

Use with a contra-angle with green colour-coding at 800 to 1,200 r.p.m.

Gates, No. 180.15... and 180.19... :

Use with a contra-angle with green colour-coding at 450 to 800 r.p.m.

Barbed broaches, No. 9107... :

Insert the broach into the root canal and rotate it through 180° - 360° to engage and withdraw the pulp.

K reamer, No. 171... :

Exert pressure during use and rotate clockwise up to 90°.

K files, No. 173... :

Filing motion combined with clockwise rotary movements up to 45°.

Hedstroem files, No. 174... :

To avoid distorting handtype instruments, they should only be used with reciprocating movements.

Root fillers, Nos. 178... and 178S... :

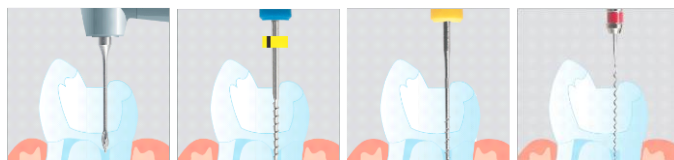
Use a green colour-coded contra-angle. Immerse the spiral (Lentulo) in the root canal sealer and, with the handpiece at a standstill, insert it as deep as possible into the root canal. Rotate the spiral at low speed (approx. 800 r.p.m.) and slowly withdraw it from the canal. The sealer remains in the canal. Product No. 178S... , Safety spiral filler on a contra-angle shank - can be inserted into the root canal flexibly.

Using Instruments for contra-angles:

When using a contra-angle, always take care and follow the manufacturer's instructions. Incorrect operation can easily cause the instrument to fracture.

Instrument coding:

Hand types are colour-coded to denote their size and standard. Hand type instruments are also marked with a symbol and size. The plastic grips of the barbed broaches are

coloured to denote their size. Spiral fillers with metal shanks are coded with coloured rings.

Servicing and storing the instruments hygienically:

Before the first application on the patient and immediately after every use, immerse the instruments in a tank full of disinfectant and detergent (admission of VAH/DGHM or FDA, respectively distinguishing mark CE). (Always follow the manufacturer's recommended mixing ratio and immersion time in the disinfectant. Do not use disinfectants containing phenylic acid). Rinse the instrument under running water and check for residues, clean it again if necessary and dry it.

Sterilization:

Please sterilize in an autoclave at 134°C/5 min. or according to the unit manufacturer's instructions. Comply with DIN EN 13060 and DIN EN 285 (sterile goods packaging according to DIN EN ISO 11607/DIN EN 868-2ff). Observe guidelines from Robert Koch Institut (Hygiene Requirements in Dentistry) or the valid legal and hygiene regulations in the respective country!

Detailed recommendations for preparation in accordance with DIN EN ISO 17664

www.edenta.com

Storage and Keeping Conditions:

In dry conditions and protected against contaminants.

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