EDENTA AG, Hauptstrasse 7, CH-9434 AU/SG – Switzerland
Tel.: +41 71 747 25 25 Fax: +41 71 747 25 50 e-mail: info@edenta.ch

EDENTA Etabl., Industriest. 13, LI-9486 Schaanwald – Liechtenstein
Tel.: +423 375 20 50 e-mail: info@edenta.com

DE

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www.edenta.com

GB

Flexi-Snap Polishers Instruction for Use:

1. Indications / Application

Single-use polishers for contouring, finishing and high-lustre polishing of composite restorations in the dental practice

The 4 different grit sizes are colour-coded.

- Green = coarse / contouring
- Blue = medium / finishing
- Red = fine / prepulishing
- White = ultrafine / high-lustre polishing

Only use the Flexi-Snap for the applications given in the instructions.

2. Instructions for use

- Remove excess composite with a tungsten carbide finisher or diamond finisher.
- Only use the Flexi-Snap with a 4038RA mandrel.
- Preset the optimum motor speed = 10 -12,000 rpm (max. motor speed = 15,000 rpm).
- Always use coolant spray during preparation (min. 50 ml/min).
- Apply light pressure when polishing, 10 – 20 s per grit size, working sequence green, blue, red, white.
- Avoid contact with soft tissue.

Caution:

Excess motor speed, too much pressure or polishing without coolant spray can overheat the composite restoration and destroy the Flexi-Snap polishers causing possible injury.

Avoid jamming or excessive bending of the Flexi-Snap polishers.

3. Cleaning instructions

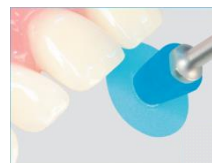
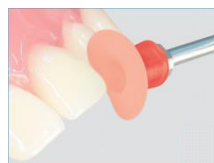
Flexi-Snap polishers are single-use products. They cannot be sterilised and should be discarded after use.

The mandrels are reusable and should be prepared again after each use:

- Disinfect and clean the mandrels. Adhere to the manufacturer's instruction for disinfecting time and concentration of the liquid.
- Rinse the mandrels with water and dry thoroughly. Never allow the mandrels to stay moist or dry over a longer period.
- Visually check the mandrels. Discard damaged mandrels and do not reuse!
- Sterilise the mandrels in an autoclave (134°C / 274°F) and then check for surface corrosion. Do not reuse corroded mandrels!

Detailed recommendations for preparation in accordance with DIN EN ISO 17664

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